



VITA Linearguide 3D-MASTER

The new simplicity in shade taking

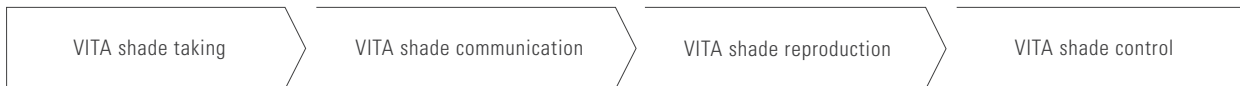

Date of issue 09.08

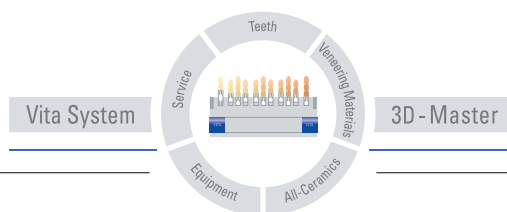

VITA

Product description

With the VITA Linearguide you can determine the correct tooth shade swiftly and accurately. The modern design and systematic structure of the VITA Linearguide enable the suitable 3D-MASTER shade to be found quickly.

The VITA Linearguide 3D-MASTER facilitates the change from conventional shade taking systems to precise shade determination and shade communication in the VITA SYSTEM 3D-MASTER. More information is available at www.vita-zahnfabrik.com

VITA Linearguide 3D-MASTER Simply efficient



Remove the VITA Valueguide 3D-MASTER from the opened Linearguide.



Make a first selection using the VITA Valueguide. This way you determine the correct level of lightness from 0 to 5.



Now make a definitive selection within the determined level of lightness from step 1 using the corresponding VITA Chroma/Hueguide.



This way you determine the correct tooth shade in a quick and reliable manner.

Advantages

- Classic linear handling
- Self-explanatory, simple use
- Quick, reliable and precise shade determination for satisfied customers
- Modern design which reflects perfect esthetics and clear arrangement

The VITA Linearguide 3D-MASTER contains the same 29 shade samples as the VITA Toothguide 3D-MASTER. It offers precise shade-taking in VITA SYSTEM 3D-MASTER according to the conventional procedure.

Benefit from the advantages of the VITA SYSTEM 3D-MASTER and the possibility to determine all tooth shades safely and systematically.

Designation	Content	Prod. No.
VITA Linearguide 3D-MASTER	1 box	B363

Publications	Designation	Order No.
VITA Linearguide 3D-MASTER	Product Data Sheet	1532E

US 5498157 A - AU 659964 B2 - EP 0591958 B1

VITA